2S<sup>®</sup>

Paramedical  
Equipment

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### ANKLE FOOT ORTHOSIS

#### SPECIFICATIONS / INTENDED USE

The AFO use is recommended for the control of the ankle position. AFOS' are designed to help patients who suffer from muscle weakness(drop foot) caused by a stroke, spinal cord injury, muscular dystrophy, multiple sclerosis and peripheral neuropathy. They also immobilize the ankle and lower leg in the presence of arthritis or fracture and enhances normal movement of the ankle, while increasing the efficiency of walking.. AFOs' (Ankle Foot Orthosis) have been used for decades to treat physical problems with the lower limb. AFO's are commonly used to treat foot drop due to Stroke, MS (Multiple Sclerosis), CP (Cerebral Palsy), Nerve Damage, Spinal Cord Injuries, Traumatic Brain Injuries, and General Weakness. The AFO can also be used for positional support for patients with Excessive Muscle Tone, Paralysis, or Congenital Deformity, and to immobilize the foot/ankle in cases of Charcot Feet, Fracture, Arthritis, or Wound Management.

**CODE: 10.05.006**

The ankle foot orthosis (AFO) is an orthosis or brace that surrounds the ankle and the lower part of the foot. AFO'S are L shaped and are externally applied to the leg and foot. The 2S AFOorthosis is a flexible L shaped AFO with the upright portion behind the calf and the lower portion running under the foot. The flexible AFO provides some dorsiflexion assistance ( e.g. weak shin muscles) to the ankle joint and it also provides relative stability to the subtalar joint. that can block or limit ankle movements and stabilize the subtalar joint, while at the same time can help in the control of the adduction and abduction of the forefoot. Is made of thermoplastic material, is extremely thin and can be worn under your shoe.

**CODE: 10.05.007**

The 2S night resting AFO boot is a thermoplastic shell which encompasses the posterior (back) third of the leg and extends from the top of the calf to the end of the foot. The AFO has a soft lining and is secured with Velcro strapping. The shell is trimmed back at the ankle to allow the foot section to move upwards. The angle between the foot and leg can be adjusted using three straps which attach to the end of the foot section, to the instep and the top of the calf section . By virtue of the adjustable straps, the foot can be aligned and held in an upward position to provide a stretch to the calf muscle to reduce or prevent contracture (tightness) in the calf. This type of brace is designed to Stretch the plantar fascia and Achilles tendon in order to help pain relief and speed healing of plantar fasciitis or Achilles tendonitis. Removable multi - position AFO. Super absorbing removable/washable liner. Enabling resting of the foot and ankle in multi-positions.

Prevents ankle plantar flexion contractures. Night Resting AFO's for stretching the calf muscles aid in the management of heel pain (plantar fasciitis) and Achilles tendonitis.

#### APPLICATION INSTRUCTIONS

##### MEASUREMENT & SIZING

Sizing according to your shoe size.

S	M	L
35-38cm	38-41cm	41-45cm

##### APPLICATION

1. Open all straps to maximum end point.

2. Slide foot in proper AFO, placing heel firmly down and back into the heel cup, in a way that it embraces your heel.

3. Fasten all straps into shell in order to provide a firm but comfortable hold of the ankle.

4. In case you are allowed by your doctor to wear it with shoe, open shoe with a shoe to maximum slackness and slide foot with AFO into shoe (shoe horns may be used).

5. Secure shoe.

6. For the night/resting AFO you follow exactly the same instructions for its application.

7. Make sure that you keep your foot aligned in an horizontal position. It is recommended that you attach the T- extension/support in order to achieve better stability and full immobilization.

8. In order to take off the supports, loosen shoe and remove from foot.
9. Loosen all straps on the orthosis and remove from foot.

### **COMPOSITION**

Thermoplastic material

Inner night AFO's removable lining : Sponge 60%, Polyester 15%, Cotton 15%, Elastane 10%

### **GENERAL WARNINGS**

- These devices will not prevent or eliminate any risk of injury. Do not over-tighten the straps. If there are any signs of swelling, pain or skin irritation, discontinue immediately and consult a medical professional.
- These supports should only be used for the intended purpose in the manner described in this manual. No liability is accepted for improper use of the product.
- Sharing of the support is not recommended.
- Please keep the products out of the reach of children.
- A clean sock must be worn under your orthosis. The sock will reduce friction, making application of the orthosis easier. It will also assist in reducing the effects of perspiration. The sock should be wrinkle free and higher than the top of the orthosis. Cotton socks are recommended. If perspiration is heavy, or the AFO is intended for 24 hour wear, then change socks during the day. Your orthosis must always be worn with a shoe or sneaker when standing or walking, unless otherwise instructed. Low top, lace-up athletic style shoes with a flat bottom work best.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your healthcare specialist without delay.
- Ensure the supports are fastened comfortably.
- Check the products' composition for possible sensitivity/allergic reaction to any of the products' materials.
- Do not apply the products directly to your skin in order to avoid rashes from unwanted or excessive sweating.
- Carefully check the foot and leg for pressure areas after each wear period. Remove the orthosis and sock. Note any redness or irritation, particularly over bony prominences. The redness should disappear with 30-40 minutes. If the redness persists, or is accompanied by skin breakdown, fluid discharge or swelling, call your doctor.

### **Daily Activities**

The garment is designed to provide comfortable support for general use.

Consult your physician regarding daily activities that you are allowed to practice, depending on your condition.

### **WASHING / CLEANING INSTRUCTIONS**

#### **For the daily use AFO:**

Close the Velcro bands and wash the product with the Velcro straps hermetically shut.

To clean the inside of your orthosis, use a wet washcloth and dry thoroughly with a towel. Rubbing alcohol has proven to work best.

#### **For the night AFO :**

#### **Follow exactly the same cleaning instructions for the plastic parts**

#### **For the inner removable lining do as follows:**

- Hand wash in water at 30°C.
- Do not wring the belt to remove water.
- This is to protect the internal components from damage.
- Do not dry clean.
- Do not use bleach.
- Do not tumble-dry
- Always dry lying flat.
- Do not dry with heat. (E.g. over a radiator). Do not iron
- Make sure that the product is 100% dried before use.

### **IMPORTANT NOTICE!**

WHILE EVERY EFFORT HAS BEEN MADE TO OBTAIN MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL.

2S DISCLAIMS ANY WARRANTY, EXPRESSED OR IMPLIED THAT THESE PRODUCTS PREVENT INJURIES.

IN CASE THE PRODUCT HAS A DEFECT PLEASE CONTACT YOUR LOCAL SUPPLIER, STORE OR PHARMACY. IN CASE OF A MANUFACTURING MALLFUNCTION YOU MAY RETURN THE PRODUCT WITH YOUR PURCHASE DOCUMENT WITHIN 15 DAYS FROM THE DAY OF THE PURCHASE.