2S[®]

Paramedical
Equipment

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Instructions for use for 2S ABDOMINAL SUPPORT BELTS

CODES: 10.07.002 / 10.01.005 / 10.01.007 / 10.01.012 / 10.01.013

EN

Instructions for use

2S ABDOMINAL SUPPORT BELTS

SPECIFICATIONS / INTENDED USE

Abdominal support belts (also known as abdominal binders) softly support your greater abdominal and lower back area, helping you get out of bed, walk and move after surgery. These belts are perfect for the first month after open abdominal surgery, hysterectomy, myomectomy, intestinal resection, appendectomy or hernia surgery.

These belts can support your greater abdominal area after c-section or/and after delivery as well as after cosmetic surgery (liposuction).

Additionally you can wear an abdominal belt in order to provide warmth in the greater abdominal and lower back area (especially the belts made of neoprene or cotton fabric), thus reducing pain from rheumatism and muscle spasms.

Provide support and heat during sports and help burn fat and thus lose weight faster in the abdomen area

Support the ribs and sides of your torso, if worn a bit higher, after rib injuries or muscle injuries and pain.

Abdominal support belts do not feature metallic or plastic supportive accessories. They are made of neoprene or elastic cotton and have a snug but gentle fit on your body, with supportive and warming effect.

APPLICATION INSTRUCTIONS




MEASUREMENT & SIZING

Proper sizing is important. Measure around your waist to determine the correct size. If you have a large mid section, consider getting the next size up so that you will be comfortable. If the belt you are using is One Size, then the fitting table below does not apply to you.

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APPLICATION

Proper application of the collar is as important as proper sizing for immobilization and patient comfort. The two-piece design consists of a FRONT piece (the piece with the opening) and a BACK (occipital) piece which are packaged as a set.

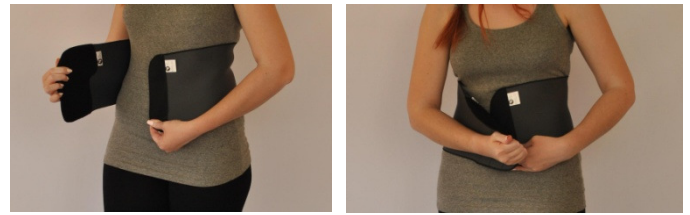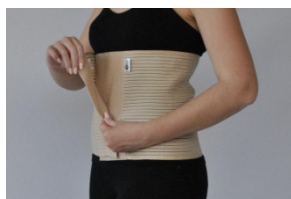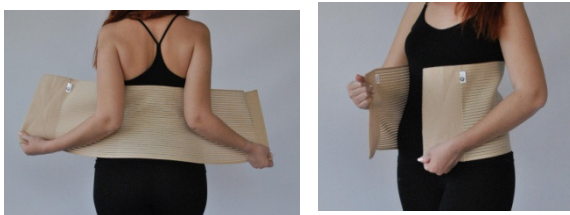- Use one hand to hold one side of the abdominal belt in position. Use the other hand to stretch the belt around the waist. Secure the belt in the front or in the side (at a spot that does not bother you with potential scars or injuries) with Velcro closure
- Do not suck your stomach in all the way, and do not let your midsection (belly) hang out all the way before putting on the belt. Start with a neutral position in which you are comfortable

- The Gibaud type abdominal belt is also offered in a non-velcro version, one you slip on like an elastic skirt. As it is seamless it will not bother your abdominal area if you have stitches or scars after an operation.

COMPOSITION

Elastic belts are made of high quality elastic suppression fabric. Cotton 30%-40%, Polyamid 35% - 45% Nylon 15%, Elastane 15%-35%

Neoprene belts are made of high quality NEOPRENE fabric, breathable, 3-4,5 mm thick, laminated with hypo allergic jersey fabric (NEOPRENE 80%, nylon 20%)

GENERAL WARNINGS

Side effects harmful to health have not been reported to date. Please consult your clinician if you have the following:

- Skin disorders or injuries in the relevant part of the body - particularly if inflammation, raised scars with swelling, redness or excessive heat buildup is present.
- Impaired sensation.
- Heart or lung conditions.

The belt should only be used for its intended purpose in the manner described in this manual. No liability is accepted for improper use of the product.

- Sharing of the belt is not recommended.
- Please keep the belt out of the reach of children. The belt is not a toy.
- Should you notice any unusual changes (e.g. increased symptoms) please contact your healthcare specialist without delay.
- Ensure the belt is fastened comfortably. If fastened too tightly, it may restrict breathing and blood flow.
- Avoid the use of ointments/ lotions or any substances including grease or acids during use of the belt.
- Check the product's composition for possible sensitivity/allergic reaction to any of the product's materials.
- In case you are allergic to Velcro or Neoprene material do not apply the product directly to your skin but over cotton underwear. It is advisable to use protection between the skin and the product in order to avoid rashes from unwanted or excessive sweating.

Daily Activities

The garment is designed to provide comfortable support for general use. It should not be used for excessive or exerting activities.

Depending on your condition (e.g. post-surgical recovery plan), you must consult your physician on which daily activities you are allowed to perform.

Driving

Do not attempt to drive if you are in a post surgery recovery period as positioning the seat belt can apply pressure on areas in your abdomen that you are not supposed to apply pressure on. If you must ride a car, seat in the back seat, in order to avoid crushing on the air bags in case they open.

Still, if your physician allows it, you can drive whilst wearing a support belt. Make sure that you are comfortable with your belt in the seating position and use a lumbar support cushion for extra support while driving.

Sleeping

You can wear your support belt while sleeping, still your physician will advice you in this matter.

Showering

You can shower wearing your belt, if your doctor allows you to shower (depending on your surgical wound type). It is best to have 2 support belts. You can wear the belt in the shower and change it over with the second, dry one, once you're out. Dry yourself and reapply the dry belt as per the instructions above.

WASHING / CLEANING INSTRUCTIONS

Close the Velcro bands, if any, and wash the product with the Velcro straps hermetically shut.

Never machine wash. Hand wash in water at 30°C. Do not wring the belt to remove water. This is to protect the internal components from damage.

Do not dry clean. Do not use bleach. Do not tumble-dry

Always dry lying flat. Do not dry with heat. (E.g. over a radiator). Do not iron

Make sure that the product is 100% dried before use.

IMPORTANT NOTICE!

WHILE EVERY EFFORT HAS BEEN MADE TO OBTAIN MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL.

2S DISCLAIMS ANY WARRANTY, EXPRESSED OR IMPLIED THAT THESE PRODUCTS PREVENT INJURIES.

IN CASE THE PRODUCT HAS A DEFECT PLEASE CONTACT YOUR LOCAL SUPPLIER, STORE OR PHARMACY. IN CASE OF A MANUFACTURING MALLFUNCTION YOU MAY RETURN THE PRODUCT WITH YOUR PURCHASE DOCUMENT WITHIN 15 DAYS FROM THE DAY OF THE PURCHASE.