2S[®]

Paramedical
Equipment

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Instructions for use for 2S ARM-SHOULDER IMMOBILIZING STRAP

CODES: 10.06.031

EN

Instructions for use

2S Arm-Shoulder immobilizing strap

SPECIFICATIONS / INTENDED USE

The Arm-Shoulder Immobilizer immobilizes the shoulder and safely positions the arm close to the body. The shoulder immobilizer you have been given is designed to support your arm, but also to protect you from doing certain motions. It is also to protect your arm from other people pulling on or against the arm and to protect the arm at night, so you do not sleep on it wrong or move it in the wrong position.

APPLICATION INSTRUCTIONS

MEASUREMENT & SIZING

This product is one size, so you do not need to apply any special measuring.

APPLICATION

Before you put on the immobilizer, separate its components, for ease of application

1. Place large chest strap around your chest, just below the nipple line. It is often easiest to apply the chest strap in the sitting position with the shoulder supported. Secure Velcro tightly around the chest. Bring the attached shoulder strap from the back over your good shoulder to the front.

2. Wrap the upper arm cuff around the biceps and secure with the Velcro on the side of the chest strap.

3. Bend your elbow to 90o and secure the wrist/hand cuff on the chest strap.

4. Adjust the shoulder strap that is attached and buckled on hand/wrist cuff. Position the small pillow on your good shoulder for greater comfort

COMPOSITION

60% soft, breathable sponge for added comfort, 15% polyester, 15% cotton, 10% elasthan

GENERAL WARNINGS

Don't continue to wear this device if it causes numbness or increased pain in your arm or shoulder. Call your doctor immediately if you experience either of these symptoms.
This device will not prevent or eliminate any risk of injury. Do not over-tighten the straps. If there are any signs of swelling, pain or skin irritation, discontinue immediately and consult a medical professional.

The support should only be used for its intended purpose in the manner described in this manual. No liability is accepted for improper use of the product.

- Sharing of the support is not recommended.
- Please keep the support out of the reach of children. It is not a toy.
- Should you notice any unusual changes (e.g. increased symptoms) please contact your healthcare specialist without delay.
- Ensure the support is fastened comfortably.
- Avoid the use of ointments/ lotions or any substances including grease or acids during use of the support.
- Check the product's composition for possible sensitivity/allergic reaction to any of the product's materials.
- Do not apply the product directly to your skin but over cotton underwear. It is advisable to use protection between the skin and the product in order to avoid rashes from unwanted or excessive sweating.

Daily Activities

The garment is designed to provide comfortable support for general use.

Driving

You should not drive while wearing this support.

Sleeping

You can wear this device while sleeping. Your physician or orthotist will advise you whether you should wear this device while sleeping.

Showering

Remove the support when showering, unless your doctor tells you to keep it on at all times.

WASHING / CLEANING INSTRUCTIONS

Close the Velcro bands, if any, and wash the product with the Velcro straps hermetically shut.

Never machine wash. Hand wash in water at 30°C. Do not wring the belt to remove water. This is to protect the internal components from damage.

Do not dry clean. Do not use bleach. Do not tumble-dry

Always dry lying flat. Do not dry with heat. (E.g. over a radiator). Do not iron

Make sure that the product is 100% dried before use.

IMPORTANT NOTICE!

WHILE EVERY EFFORT HAS BEEN MADE TO OBTAIN MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL.

2S DISCLAIMS ANY WARRANTY, EXPRESSED OR IMPLIED THAT THESE PRODUCTS PREVENT INJURIES.

IN CASE THE PRODUCT HAS A DEFECT PLEASE CONTACT YOUR LOCAL SUPPLIER, STORE OR PHARMACY. IN CASE OF A MANUFACTURING MALLFUNCTION YOU MAY RETURN THE PRODUCT WITH YOUR PURCHASE DOCUMENT WITHIN 5 DAYS FROM THE DAY OF THE PURCHASE.