2S<sup>®</sup>

Paramedical  
Equipment

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
Instructions for use for  
**2S RIGID CERVICAL COLLARS (with or  
without chin support)**

CODES: 10.00.001/10.00.002

**EN**

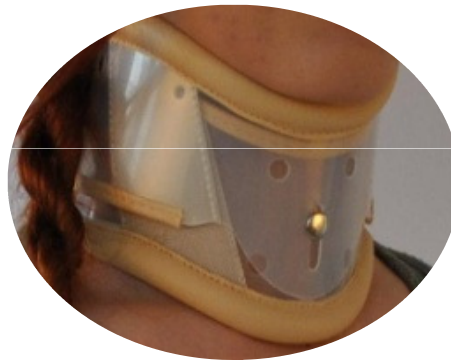## Instructions for use

# 2S RIGID CERVICAL COLLARS (with or without chin support)

### SPECIFICATIONS / INTENDED USE

Ventilated Polyethylene Collar constructed in two sections to enable height adjustment  
 Padded borders with vinyl leather covering for increased comfort  
 Secure hook and loop closure fastening to the rear  
 Post trauma e.g. whiplash, post surgery, longer term users e.g. R/A and O/A  
 Neck Pain e.g. sprains and strains where more support than a soft collar is required

### APPLICATION INSTRUCTIONS

#### MEASUREMENT & SIZING

Proper sizing is important for patient immobilization and comfort. Always maintain your head in neutral alignment. Circumference: Measure around your neck. The measurement corresponds to the size CIRCUMFERENCE of the collar.

S	M	L	XL
34-36cm	36-38cm	38-40cm	40-42cm

#### APPLICATION


*Rigid Cervical Collars are made of 2 pieces.* The depth and height of the collar can be adjusted by separating the two sections and re-positioning them.


1. Hold the collar in both hands

2. Place the collar underneath your chin and wrap it around your neck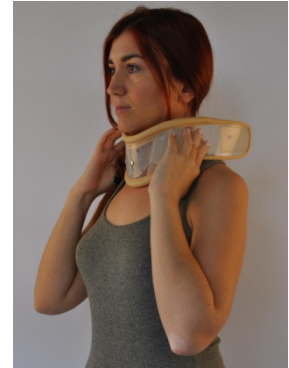

3. Fasten the hook and loop strap firmly around the back of your neck ensuring the collar is not too tight to restrict breathing or circulation

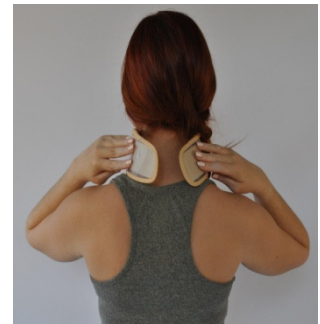The collar should fit snugly between your collar bone and chin

#### COMPOSITION

Ventilated Polyethylene 90%, vinyl leather 10%

#### GENERAL WARNINGS

- Check daily for redness or breaking skin on your chin, collarbone, and breastbone and get someone to check the back of your head. Such irritations can occur if the collar is not fitted correctly. Rashes can occur and often they are caused by sweat building up. Therefore more regular collar changes and washing of the collar may be required.
- **CAUTION:** Talcum or substitute powders and creams should not be used on skin underneath a collar

#### Contact your caregiver if:

- You have back pain.
- You have signs of a pressure sore under the collar. This would include a red, painful, or open area of skin.
- You have chest pain or trouble breathing that is getting worse over time.

#### Seek care immediately if:

- You feel the collar is too tight or too loose. Lie down flat after contacting your caregiver. Your collar may need to be checked.
- You suddenly feel lightheaded and have trouble breathing.
- You have new and sudden chest pain. You may have more pain when you take deep breaths or cough.
- Your arm or leg feels warm, tender, and painful. It may look swollen and red.

## Daily Activities

### Driving

It is strongly advised that you do not drive whilst wearing a rigid collar as it is extremely dangerous. The collar resulted immobilization combined with neck stiffness will restrict your vision making driving unsafe.

### Sleeping

It is not recommended to wear your collar during sleep. Instead it is recommended you sleep only with a rolled towel or a small pillow positioned behind your neck and not with a normal pillow. A normal pillow may cause a pressure sore (breaking skin) on the back of your head. If this occurs, call your orthotist.

### Walking

When walking you may have difficulty in looking down. This may be of concern when there are stairs or uneven surfaces such as concrete. It is best to hold onto a handrail when walking down stairs.

### Eating and drinking

You may find that eating and drinking can be difficult at first as you will have difficulty looking down at your plate or tilting your head back to drink. Try and hold your plate close to your mouth to avoid any spills and use a straw to drink.

### Showering

It is best to have 2 collars. Wear the collar in the shower and change it over with the second collar once you're out. You can change the collar on your own; however, a second person to assist your collar change is helpful. It is best to change the collar in front of a mirror either standing up or sitting with your feet flat on the floor. Ensure that you keep your head very still when you remove the wet collar after your shower. Dry your neck and reapply the dry collar as per the instructions above. It is advised that males should shave daily after showering while the collar is off. Be sure to keep your head very still.

### WASHING / CLEANING INSTRUCTIONS

Wash periodically by hand with a damp cloth and mild soap. Use a towel to absorb most of the dampness and then allow to dry at room temperature. Do not machine wash, hang up, or iron and do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight etc. Do not expose to alcohol, ointments or solvents.

### **IMPORTANT NOTICE!**

WHILE EVERY EFFORT HAS BEEN MADE TO OBTAIN MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL.

2S DISCLAIMS ANY WARRANTY, EXPRESSED OR IMPLIED THAT THESE PRODUCTS PREVENT INJURIES.

IN CASE THE PRODUCT HAS A DEFECT PLEASE CONTACT YOUR LOCAL SUPPLIER, STORE OR PHARMACY. IN CASE OF A MANUFACTURING MALLFUNCTION YOU MAY RETURN THE PRODUCT WITH YOUR PURCHASE DOCUMENT WITHIN 5 DAYS FROM THE DAY OF THE PURCHASE.